

KYDA Leather & Sheepskin

MEDICAL SHEEPSKIN SPECIALISTS

ABN 44 358 784 954

HOSPITAL GRADE MEDICAL SKIN

Medical rugs are prepared from specially selected sheepskins which are treated according to the standards for Medical sheepskins.

These rugs have very dense and springy wool fibres which support the body and allow air to flow freely. This keeps the person surprisingly cool in summer and warm in winter. The rug can absorb up to 30% of its weight in water without feeling wet. These features make the rug very comfortable and help to prevent bed sores.

Medical sheepskin rugs are therefore ideal for persons who are required to spend long periods sitting or lying down, as well as for babies and infants.

**100% AUSTRALIAN
SHEEPSKIN**



MACHINE WASHABLE

CLEANING INSTRUCTIONS

- If the rug is not very soiled:
 - sponge down using warm water and pure soap
 - sponge rinse
 - shake rug to remove excess water
 - dry flat (fleece uppermost) or on a line out of direct sunlight
 - shake rug vigorously to raise fibres
- If the rug needs to be fully laundered (about every 3 to 4 weeks)
 - using pure soap (or a recommended wool detergent), wash in a machine on the wool cycle using warm water.

Note:

The rug can also be washed using water up to 60° C and a cleaning agent such as Woolskin Sheepskin Shampoo & Woolwash - or, gently hand wash in warm water for not more than 5 minutes. Rinse twice in lukewarm water, 3 minutes each time. Spin or shake vigorously to remove excess water.

- do not wring, shake rug vigorously to raise the wool fibres.
- dry flat in the shade (or on a line), or tumble dry on low heat.
- Shampoo available from your supplier or direct from **KYDA Leather & Sheepskin**.
- If the rug has been soaked with urine, it should be rinsed With cold water as soon as possible, then laundered as above.
- THIS RUG MAY BE DRY-CLEANED.
Specialist suede and leather cleaners generally include re-texturing with cleaning to reduce any loss of softness. Air well after dry-cleaning.

REGULAR

LARGE

X-LARGE

**KYDA LEATHER
& SHEEPSKIN**

HEAD OFFICE
20 Yuruga Close, Daruka NSW 2340
Phone: (02) 6767 1230 - Fax: (02) 6767 1232
Email: kyda@optusnet.com.au
Website: www.kydaleather.com